

Checklist for NEURO PRAX® TMS/tES, NEURO PRAX® MR, NEURO PRAX® EEG

	Pelicase shipping case large model (black)
	Pelicase shipping case medium model (black)

NEURO PRAX® Therapist

	NEURO PRAX® Therapist
	Vesa adapter
	knob
	stand
	power cord (2-pole)
	keyboard (if available: USB adapter)
	mouse (if available: USB adapter)

NEURO PRAX® Trainer

	NEURO PRAX® Trainer
	Vesa adapter
	knob
	stand
	connection cable Therapist-Trainer

Amplifier

	NEURO PRAX® Amplifier
	adaptor box 32/64
	ribbon cable/s
	optical cable/s
	external charger (Mascot) with/without power cord (2-pole)

Accessories

	Trigger Module
	other:

NEURO PRAX® - Koffer

Please fill the large shipping case with the NEURO PRAX® Therapist, adaptor box, power cord, mouse, keyboard, optional accessories and consumables.

Box the medium shipping case with the amplifier, ribbon cable/s, optical cable/s and the external charger.

Pelicase shipping case large model




Pelicase shipping case medium model



A detailed manual how to place the components in the shipping cases is listed in the User Manual chapter: Appendix – Transport Packaging.

NEURO PRAX® Therapist

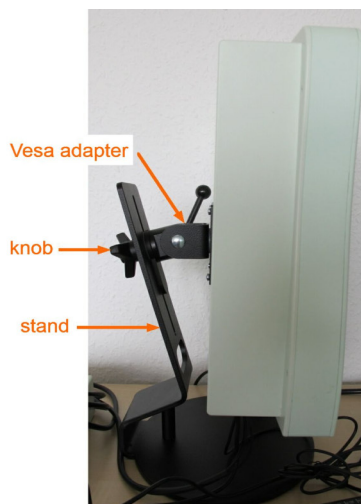

Attention!

Please mind the type label! You can only distinguish between Therapist and Trainer with the help of the type label and the sockets at the bottom side.

The Vesa adapter include all parts which are screwed at the Therapist. A locking ring is placed between Vesa adapter and stand.

Knob and Vesa adapter are unscrewable. Please mind to always enclose all parts!



Power supply

The power supply of the Therapist occurs via power cord (2-pole).



Keyboard and mouse

If available please always enclose the USB-PS2 adapter.



NEURO PRAX® Trainer

Attention!

Please mind to enclose also all parts of the Trainer: Vesa adapter, knob and stand (see NEURO PRAX® Therapist).

Connection cable Therapist-Trainer



NEURO PRAX® Amplifier

The NEURO PRAX® Amplifier consists of:

- amplifier
- adaptor box
- ribbon cable/s
- optical cable/s
- external charger Mascot with power cord (2-pole)

Amplifier



Adaptor box 32/64

adaptor box 32



adaptor box 64



Ribbon cable

The adaptor box is connected to the amplifier via ribbon cable/s.



Optical cable

The optical cables are not fixed at the NEURO PRAX[®] amplifier and can be disconnected.

Please always enclose all optical cables (see image below).



External charger

The Amplifier includes the external charger Mascot with power cord (2-pole). The batteries are built-in.



Accessories

The accessories of the NEURO **PRAX**[®] systems vary in a wide range. Please mind to enclose all accessories provided by neuroConn.

Trigger Module

Please mind to send the Trigger Module with all optical cables, USB connection and the charging device.

